

# enovate

M E D I C A L

## MobiusPower Scout User's Guide




3/12/2020

## Download

- MobiusPower Scout can be downloaded from link below

<http://mobiusscout.myenovate.com/>


## Home screen

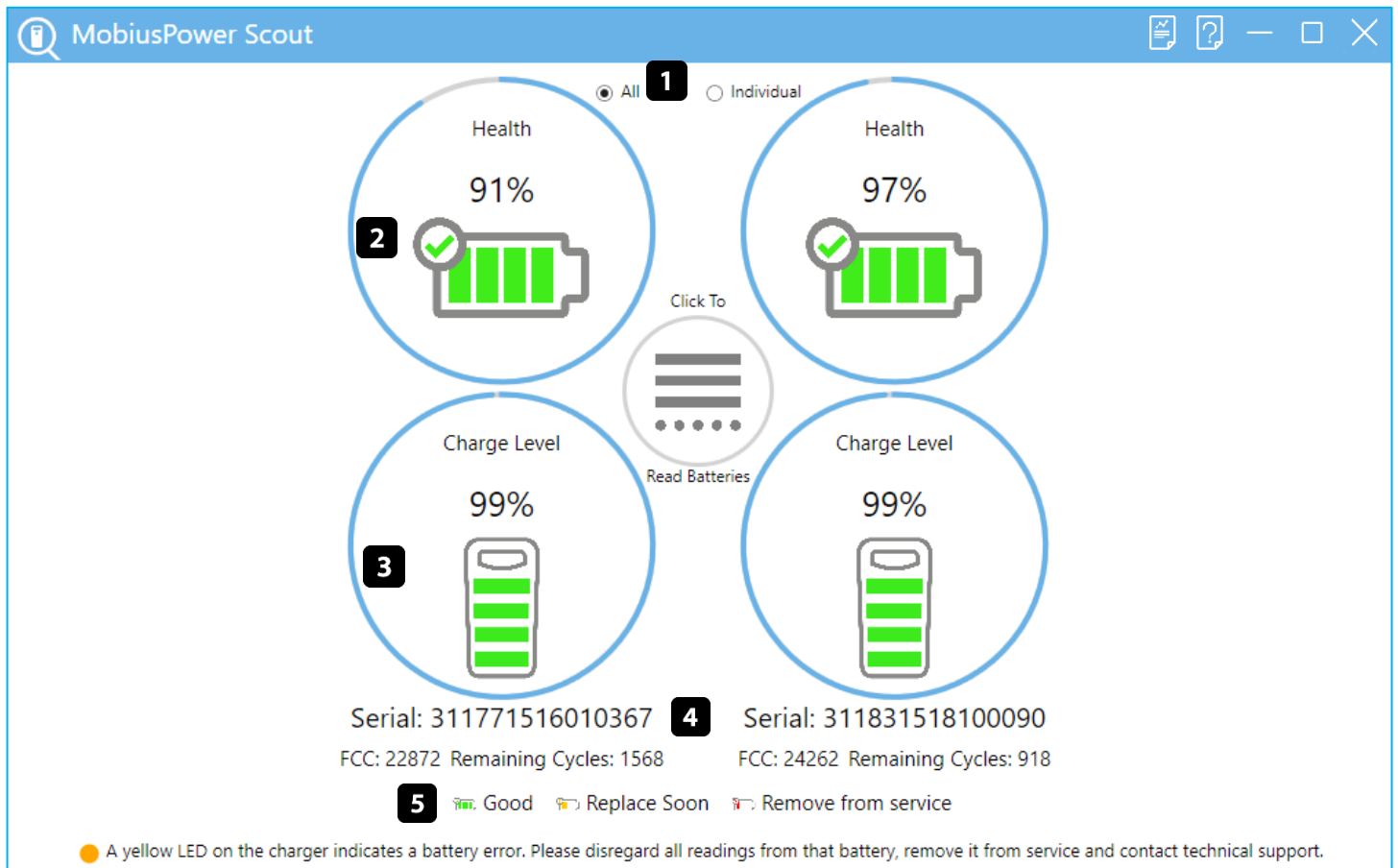


1. Read Mode Selector
2. Battery Health
3. Battery Charge Level
4. Battery Serial Number, FCC, Remaining Cycles
5. Legend
  - a. 71% and up health and/or over 270 cycles remaining: Green (Good)
  - b. 70% - 61% Health and/or 91 – 269 cycles remaining: Yellow Warning (Replace Soon)
  - c. 60% and below Health and/or 0 – 91 cycles remaining: Red Critical (Remove from Service)

# Reading Batteries



1. Open MobiusPower Scout and login

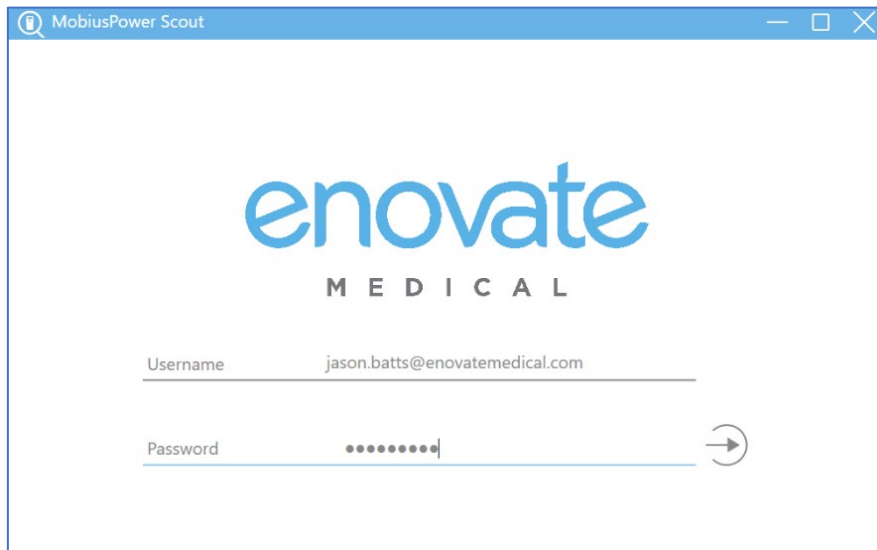


2. Power on the charger and connect the charger to an open USB port on the PC.
  - a. Scout will automatically detect a two bay or four bay charger. The Home Screen will update to reflect the number of bays automatically. If Scout does not detect a charger, "Disconnected" will appear at the top of the screen

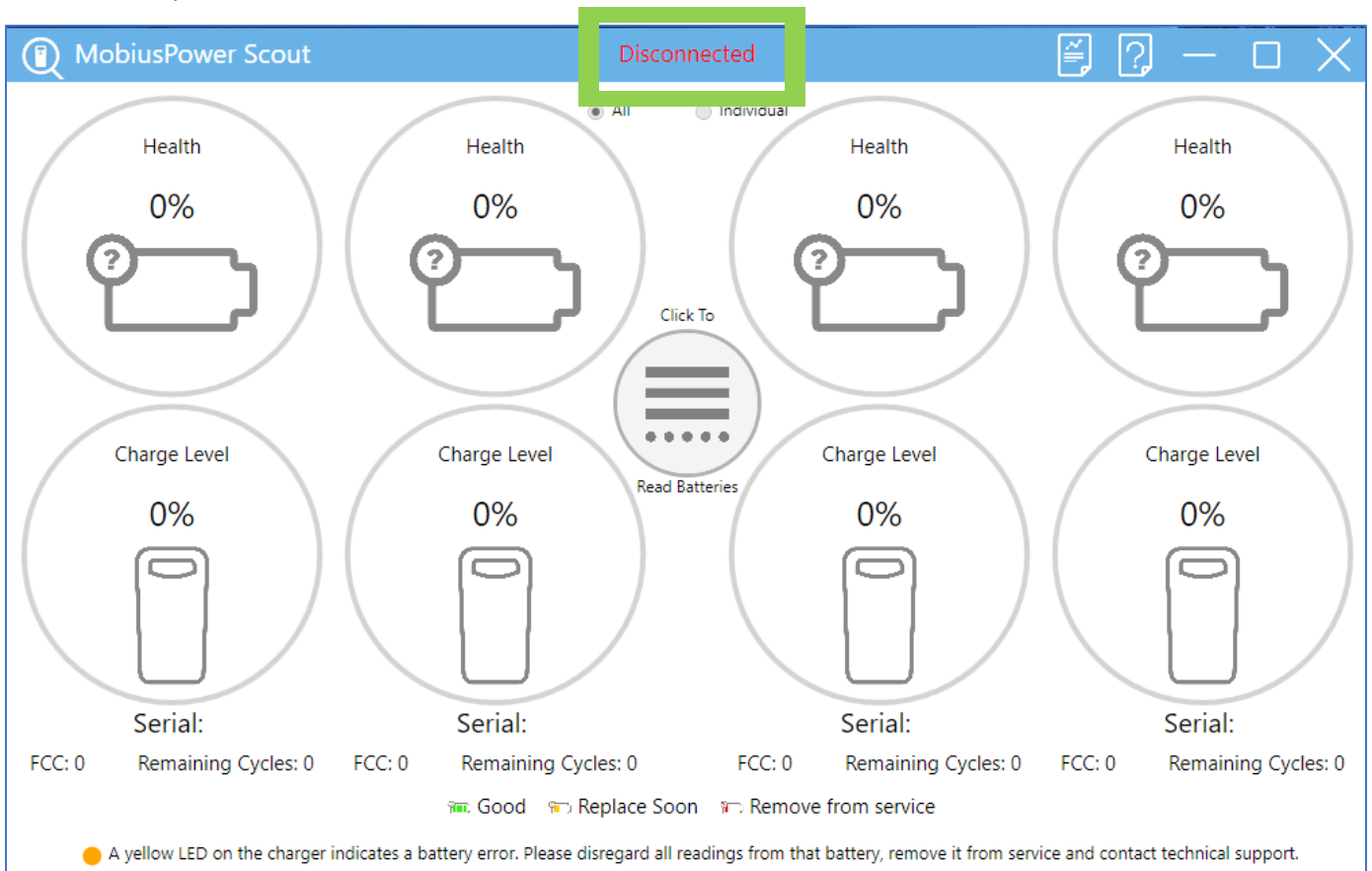


3. Insert the battery or batteries into the charger. Wait for the serial numbers to display on the screen.
  - a. If you are in ALL Mode, select “Click To Read Batteries” to read all bays simultaneously
  - b. If you are in Individual Mode, select the bay number associated with the battery you wish to read.

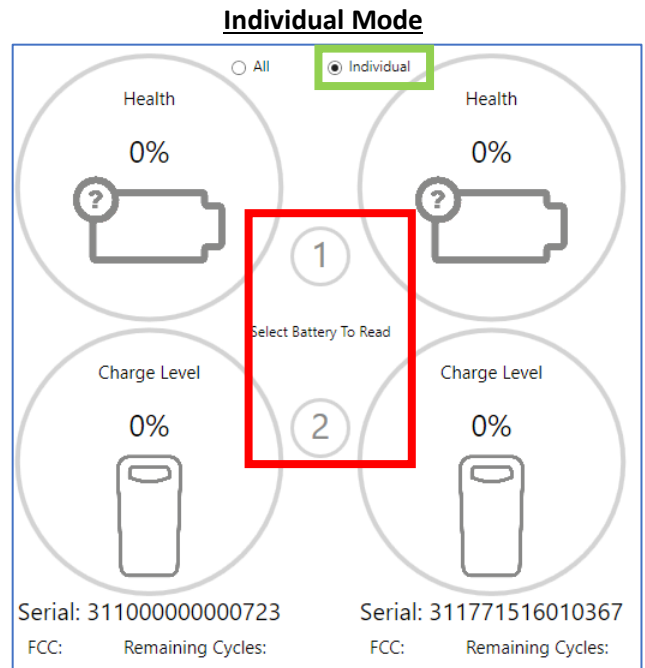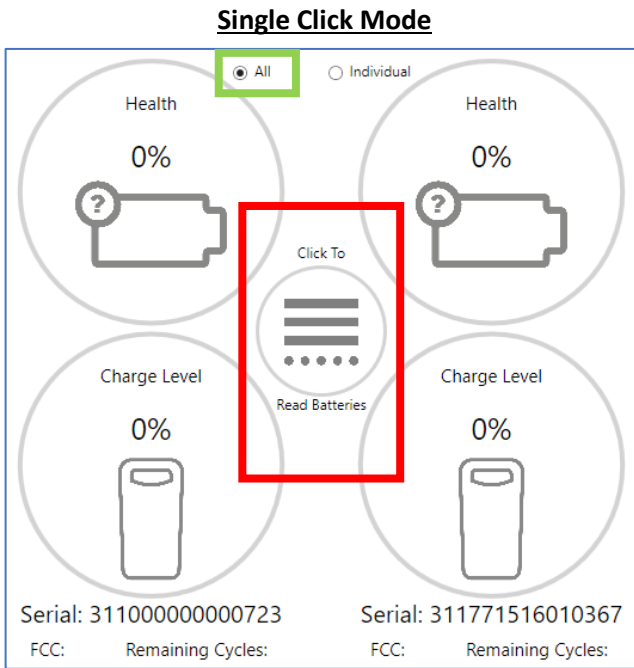


4. A successful read will return the battery’s health, charge level, FCC, and Remaining Cycles.

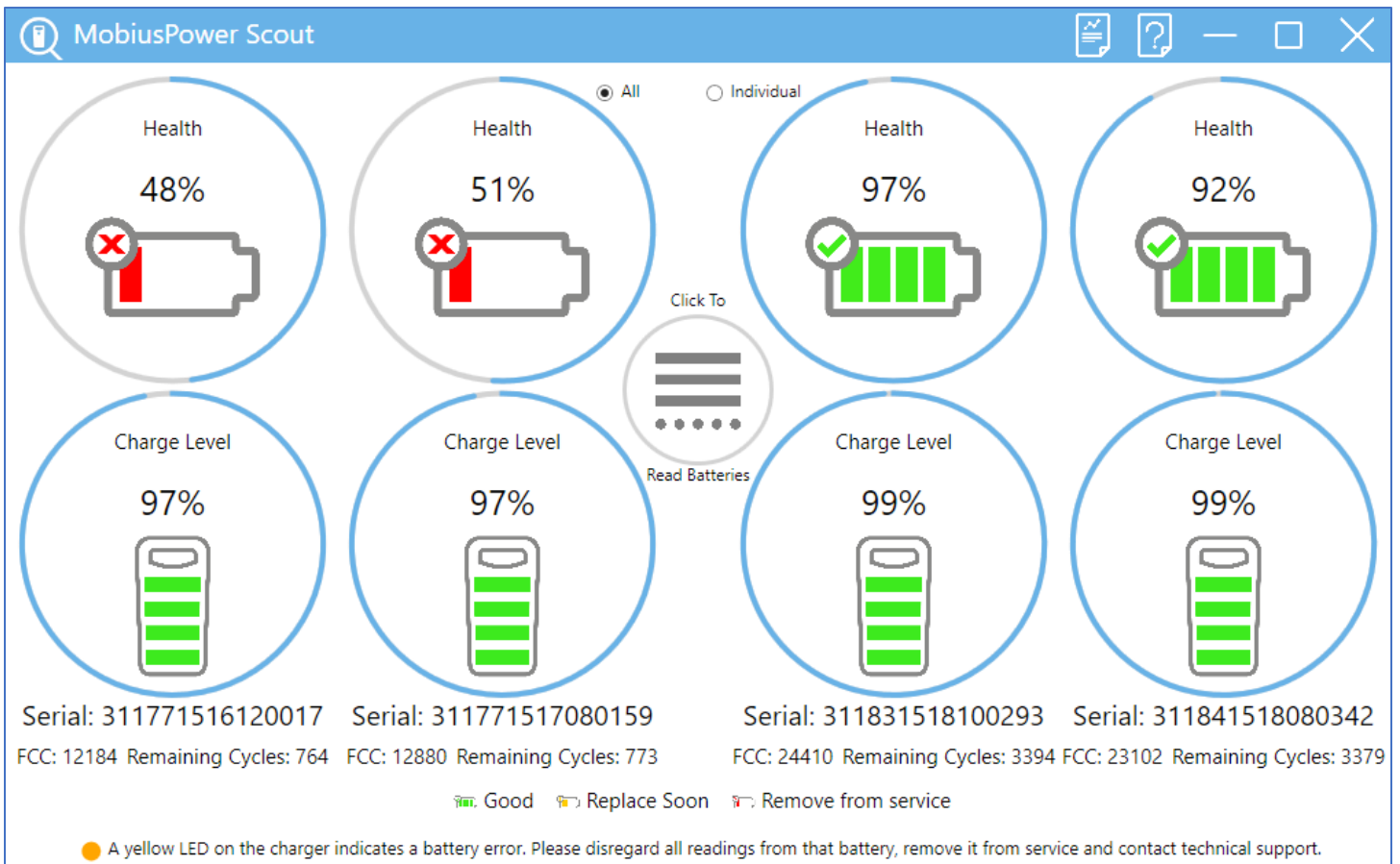

- a. In order to ensure an accurate health reading, the battery must be charged 90% or above. If a battery is below 90%, Scout will display the following message

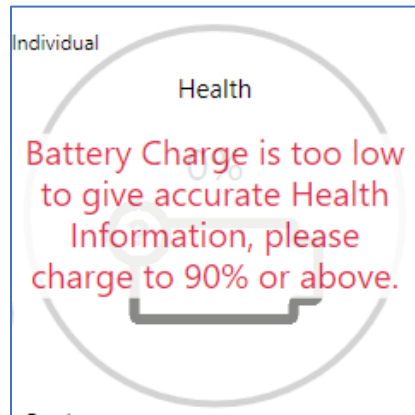


- b. If Scout encounters a USB Read Error during a reading, Scout will indicate an error has occurred and instruct the user to retry after 15 seconds.

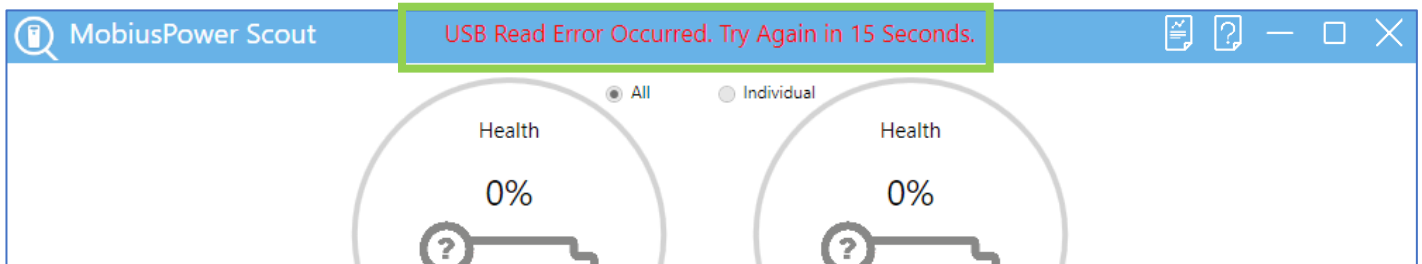

5. When switching batteries after a successful read
  - a. Remove the battery (or batteries) from the bay charger.
  - b. Wait until the serial number(s) are cleared from the screen (could take up to 30 seconds)
  - c. Insert next set of batteries into the charger.
  - d. Wait until the serial numbers of the batteries are displayed.
  - e. Read batteries as described in step 3.

- To copy an individual serial number, right click on the serial number and select “copy”.

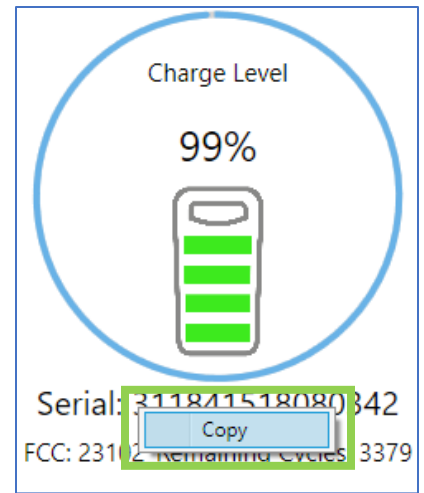

- To export the reading to a log file, click on the export button after a successful reading.

